



## **Phonak Roger Phone Adapter**

Part code: A-C-PHA



### **Description**

Use in conjunction with a Phonak Roger hearing system and a corded telephone for easy telephone conversations.

With this adapter your phone conversations from your corded phone can be fed directly into your hearing aids via your Phonak Roger Pen. This 'direct audio input' straight into your ears makes phone conversations far easier.

### **Mode of Operation**

When this equipment is set up you pick up the handset of your desk phone as usual to make a call and the audio from the telephone is wirelessly transmitted directly into your hearing aids, resulting in improved audibility of telephone conversations.

You can still use the Roger Pen around the office, and if your phone rings simply drop the Roger Pen into the charging slot and pick up the phone. The audio from the phone will start streaming into your hearing aids directly.

If you want to swap back to microphone mode quickly simply lift the Roger Pen out of the docking station.